



The  
precision  
expert

## CLEANING INSTRUCTIONS

Orfit immobilization devices can be cleaned and disinfected by means of an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Do not soak.**



Never use aerosol sprays, corrosive cleaning agents, solvents, abrasive detergents or cleaning agents containing bleach.



For more detailed cleaning instructions, [watch this video](#).





The  
precision  
expert

## OVERVIEW OF TESTED CLEANING AGENTS

The below cleaning agents have been tested on low density head supports and The AIO Solution 2.0 cushions (Low Density), regular density head supports (Regular Density), the AIO Solution 3.0 cushions (AIO 3.0) and the SBRT cushion.

Cleaning Agent	Low Density Head Supports and AIO 2nd Generation	Regular Density Head Supports	AIO 3.0	SBRT Cushion
AF Wipes	✗	✓	Not tested	Not tested
Alklanet	✗	✗	Not tested	Not tested
Aniosurf ND Premium	✓	✓	✓	✓
Aniospray Quick	Not tested	✓	Not tested	Not tested
Bacillol 30 Tissues	✗	✓	Not tested	Not tested
Cidalkan wipes	✗	✗	✓	✗
Cidalkan spray	✗	✗	✓	✗
Clinell Detergent Wipes	✓	✓	✓	✓
Clinell Sporicidal Wipes	✓	✓	✓	✗
Clinell Universal Spray	✗	✓	✓	✗
Clinell Universal Wipes	✗	✓	✓	✗
Cleanisept wipes forte maxi	✓	✓	✓	✗
Descosept sensitive wipes	✓	✓	✗	✗
Fugaten-spray	Not tested	✓	Not tested	Not tested
Hospisept-Tuch	Not tested	✓	Not tested	Not tested
Hydral 70	✓	✓	✓	✓
Incidin Alcohol Wipe	✓	Not tested	✓	✗
Incidin Foam	✗	✗	Not tested	Not tested
Incidin Liquid	✗	✗	✗	✗
Incidin Oxifoam	✗	✗	✗	✗
Incidin Oxyfoam S	✗	✗	✗	✗
Incidin Oxywipes	✗	✗	✗	✗
Incidin Oxywipes S	✗	✗	✗	✗
Incidin Plus	✓	✓	Not tested	Not tested
Mediwipes DM	Not tested	✓	Not tested	Not tested
Mediwipes Plus	Not tested	✓	Not tested	Not tested
Mikrozid AF Liquid	✗	Not tested	Not tested	Not tested
Orphi Ethanol 80%	Not tested	✓	Not tested	Not tested
Oxivir Excel Wipe CE	✗	✓	✓	✗
Oxivir TB	Not tested	✓	Not tested	Not tested
Sani-Cloth AF3	Not tested	✓	Not tested	Not tested
Sani-Cloth Bleach	✗	✗	✗	✗
Sani-Cloth Plus	✗	✓	✓	✗
Sani-Cloth Super	✗	✓	✓	✗
Sensitive Wipes	✓	✓	Not tested	Not tested
Surfa Safe Premium	✗	✓	✓	✗
Surfanios Premium	✗	✗	✓	✗
Trionic	✓	✓	✓	✓
Velox Foam Extra	✓	✓	Not tested	Not tested
Virusolve +	✓	✓	✓	✓
Wecoline Clean 'n Easy Interior Cloths	✗	✓	Not tested	Not tested
Wip'Anios	✓	✓	✓	✗

Orfit is able to test cleaning agents that are not represented in this table.

[www.orfit.com](http://www.orfit.com)



**ORFIT INDUSTRIES**

Vosveld 9A | B-2110 Wijnegem | Belgium  
T (+32) (0)3 326 20 26  
welcome@orfit.com

**ORFIT INDUSTRIES AMERICA**

810 Ford Drive | Norfolk, VA 23523  
T 516-935-8500  
welcome@orfit.com